

Celliss

GHAR

TOUCHLESS



GHARLEN

OBJECTIVES

> REDESIGN your silhouette

> TONE abdominals

> SMOOTH skin's texture

> REJUVENATE complexion

> REDUCE orange-peel aspect
> SCULPT buttocks
> SLIM AND SHAPE thighs
> BOOST lightness

Celliss: the slimming revolution

In cooperation with our partner DMS Group, proven experts in the medical diagnostic imaging field, we are pleased to present "CELLISS", an innovative body contouring device that doesn't even need a therapist to fight Cellulite and help to reduce weight.

This completely unique slimming machine based on a new treatment method, Cellumassage, relies on the patented "Total Slimming System" (T.S.S.) technology, a non-invasive aspiration/percussion process that simultaneously treats thighs and hips as well as buttocks or abdominals (depending on the client's position). Gharieni CELLISS is a powerful tool designed to satisfy all your requirements. Via the intuitive touch screen interface, the treatment program can be personalized. Protocols can be accessed with one quick touch, and once the treatment is launched, it does not require any intervention. The integrated protocols were developed by health/ beauty/sport professionals and allow operators to adjust the Cellumassage according to the selected treatments and the client's profile. Easy to use, this high technology device is remarkably efficient.

Celliss Innovation

ANTI CELLULITE SOLUTION SIMULTANEOUS ACTION MULTIZONE APPLICATION PERSONALIZED TREATMENTS OPERATOR INDEPENDENT NON-INVASIVE EASY TO USE

> • Gharieni CELLISS brings a "whole" new approach to slimming treatments. The "Total Slimming System" (T.S.S.) technology allows CELLISS to treat "whole" zones at the same time. With our sophisticated intuitive software tool, operators have access to personalized programs adaptable for each client's individual needs.

Ce iss Technologie

• The Cellumassage method, featured on the CELLISS, provides the ONLY solution capable of treating a significant surface of the body, simultaneously. The adjustable and adaptable aspiration/ percussion cycle, with variable intensities and speeds, massages the skin and muscles in three dimensions. The Gharieni CELLISS optimizes treatment goals through the complete, "whole" approach of its treatment programs. Stimulating and treating such a large zone at the same time makes the overall effect of the treatment more efficient. Orange-peel is reduced, skin radiance and firmness are improved, the silhouette is reshaped and thighs, abdominals and buttocks are toned.






HOW DOES IT WORK?


- 1 The aspiration stretches and moves skin giving a 'lightness' sensation.
- 2 The percussion triggers a mechanical shockwave and compresses fat, while stimulating the body's natural detoxification process.
- 3 The combination aspiration/percussion over such a large surface at the same time optimizes the treatment's effects.




MECHANICAL ASPIRATION

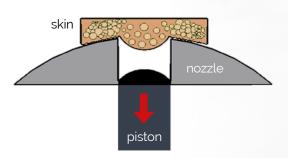

• The adipocytes are removed and stretched by the waves of mechanical aspiration.

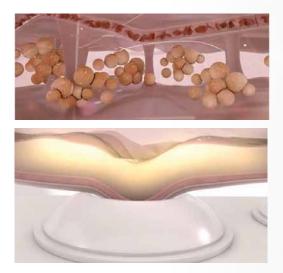

MECHANICAL PERCUSSION

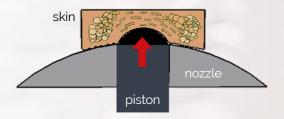

• The adipocytes are disunited highly compressed by the waves of mechanical percussion.



MULTIPLE APPLICATIONS

• The TSS (Total Slimming system) is the DNA of the Gharieni Celliss. 96 pistons – each one 3 cm (1.2 in) – take action on the adipose and muscular tissues of the buttocks, abdominals or hips with a series of noninvasive aspiration/percussion movements, whose intensity and speed can be adjusted. This unique process is controlled by the Celliss software. The combination of aspiration and percussion effects on a large stimulated area is the foundation of the Cellumassage.



Scan QR code & watch video



- Sessions typically last 30 minutes. According to the goals and the programs selected during the diagnostic phase, the treatments include 1 to 10 customized sessions.
 - Different applications for different targets
 - Safe and natural treatment allowing continuity thanks to the different programs
 - Combination of programs to correspond to client's profile and needs

Celliss Results




• 38 years old - 73 kg (160 lbs) - BMI 27,5

N° of sessions: 28

Program(s) used: Anticellulite/Slimming/Firmness

Belly -11,9 cm (-4.7 in) Hips -10,4 cm (-4.1 in) Right thigh -6 cm (-2.4 in) Left thigh -5,9 cm (-2.3 in)

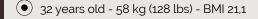

 N° of sessions: 10

Program(s) used: Anticellulite

Right thigh -2,5 cm (-0.98 in) Left thigh -2,4 cm (-0.94 in)



Celliss Effects: Key Benefits

 Reduce subcutaneous fat and improves appearance of the skin

• Stretch mechanically fibroblasts boost production of skin proteins, improve firmness & tonicity

• Clears the pressure and restores optimal flow on the vascular and lymphatic systems



Prepare muscles and provide recovery





Celliss Design



An exceptional slimming technology designed for comfort and efficiency

The Gharieni CELLISS is a powerful tool designed to meet the requirements of tomorrow's spa / beauty / wellness centers. With an exceptionally luxurious and contemporary design featuring delicate and soft-touch materials as

ONE TOUCH TECHNOLOGY TO ENHANCE PROFESSIONAL EXPERTISE

Via the intuitive touch screen interface of the CELLISS, treatment programs can be personalized for each client. Protocols can then be accessed and launched with one quick touch. The integrated protocols were well as neutral and warm colors, CELLISS naturally integrates into relaxing, zen spaces. The design was elaborated taking into consideration space constraints, but not compromising on comfort or treatment efficiency.

developed by health/beauty professionals and allow operators to adjust the intensity, frequency and amplitude of the noninvasive, painless T.S.S. (Total Slimming System) aspiration/percussion process.





Moers, Germany | Sales + Showroom // Gharieni Group GmbH • Gutenbergstr. 40 • D-47443 Moers Moers, Germany | Distribution + Service // Gharieni Group GmbH • Am Schürmannshütt 24 • D-47441 Moers Tel: +49 - (0) 28 41 - 88 300 - 0 • Fax: +49 - (0) 28 41 - 88 300 - 333 • www.gharieni.de • info@gharieni.com

> Ettlingen, Germany // Gharieni Süd GmbH • Siemensstr. 11 • D-76275 Ettlingen Tel: +49-(0)7243-3322-363 • Fax: +49-(0)7243-3322-391 • ettlingen@gharieni.com

Nederweert | BeNeLux // Gharieni Nederland BV • Hulsenweg 14 • NL-6031 SP Nederweert Tel: +31 (0) 495 - 63 30 36 • Fax: +31 (0) 495 - 62 66 63 • www.gharieni.nl • info@gharieni.nl

Dubai | UAE // Gharieni Middle East FZ-LLC • Park Hyatt Hotel • Dubai Creek Club St., P.O. Box 333203 • Dubai Tel: +971 (0) 4 602 1819 • www.gharieni.ae • info@gharieni.ae

Aventura | USA // Gharieni USA Inc. • JW Marriott Miami Turnberry Resort & Spa • 19999 West Country Club Drive Aventura Florida 33180 • Tel: +1 (855) 378 4772 • www.gharieni.com • info@gharieni.com